# Flip Up Grab Bar

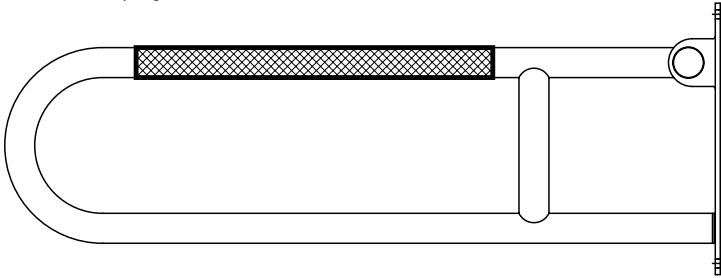
## Barre d'appui rabattable

### Pasamanos plegable



R8960/ R8962

Flip Up Grab Bar  
Barre d'appui rabattable  
Pasamanos plegable

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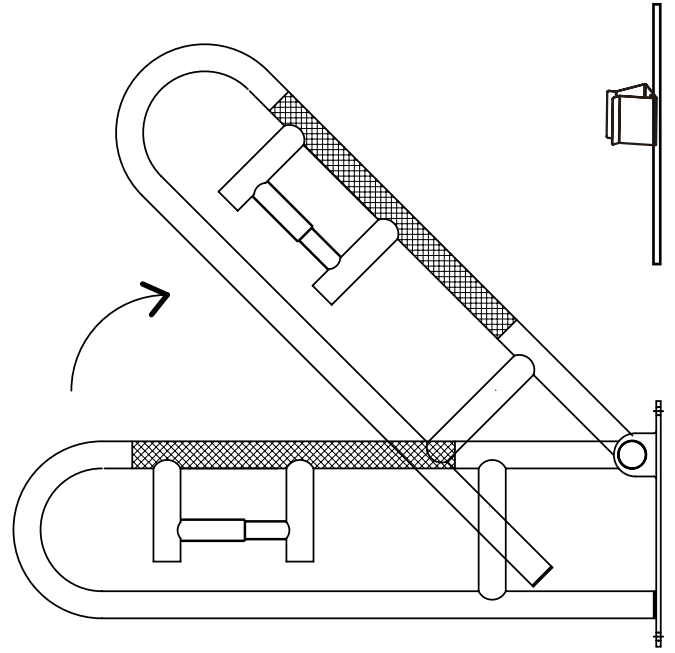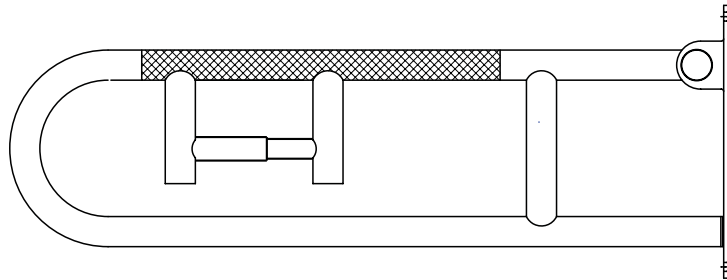Thank you for purchasing this Moen® Home Care® product. Moen® Home Care® is a leader in grab bars, offering top-quality products that are attractive and innovative, with performance features like a 300 lb. weight pull capacity that exceeds ADA requirements.

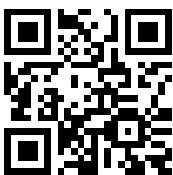
The entire Moen® Home Care® family of products offers people new options for independent living by providing outstanding comfort, convenience and style.

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**Don't forget to register and review your product** by scanning the code or visiting [moen.com/product-registration](https://moen.com/product-registration).

Download reader at [getscanlife.com](https://getscanlife.com).

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

**1**

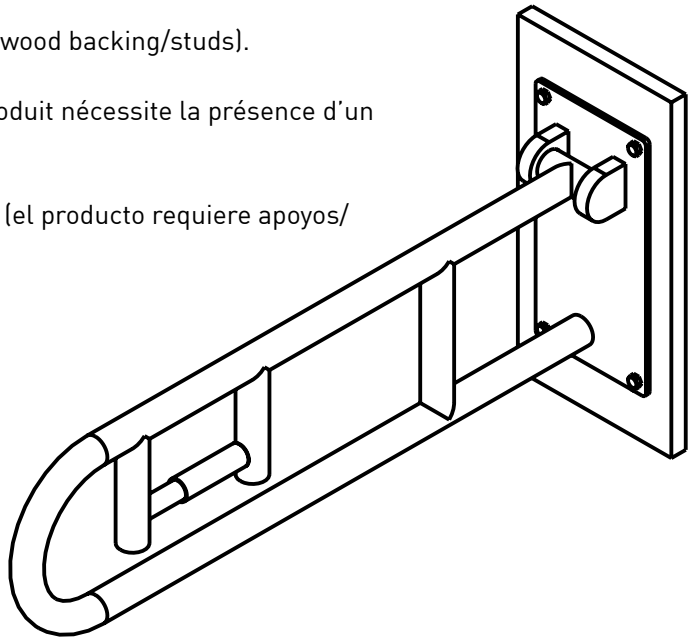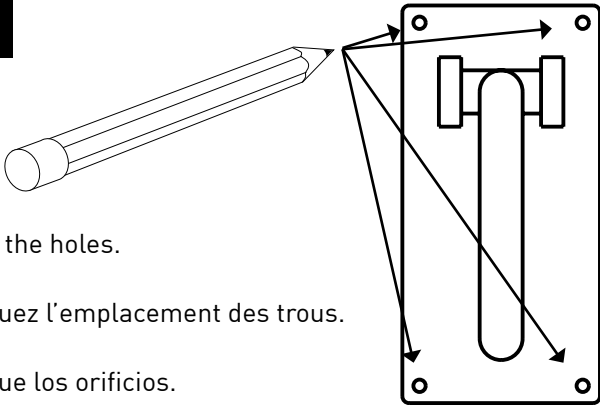Place flip up grab bar against wall (product requires wood backing/studs).

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**4**

Drill 1/4" pilot holes for hex wood screws.

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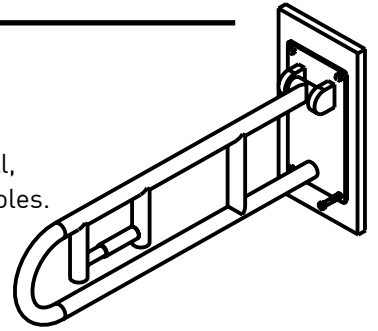
**5**

Place product back on wall,  
lining up the pre-drilled holes.

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
**6**

Secure product in place with hex wood screws.

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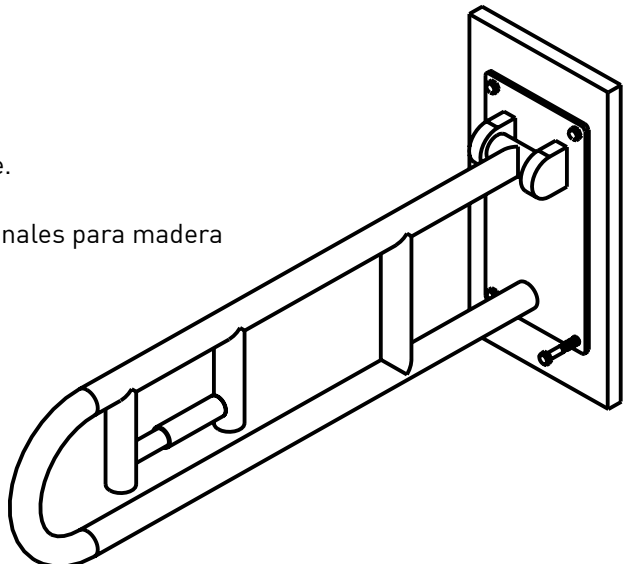
**7** With grab bar in down position, place bracket on top grab bar 20" from wall plate.

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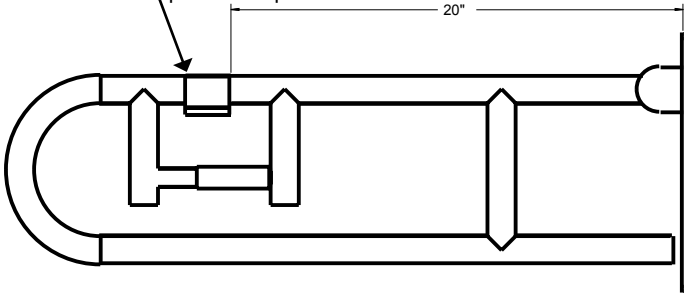
**8**


Flip up the grab bar to vertical position.

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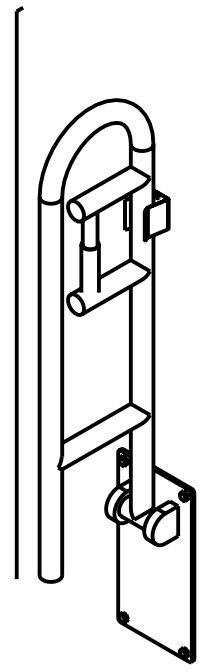
**9**

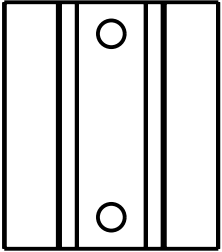Mark location of bottom of the bracket.

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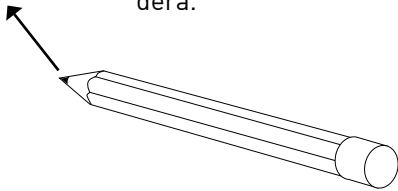**11**

Place bracket on bottom marked line.

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
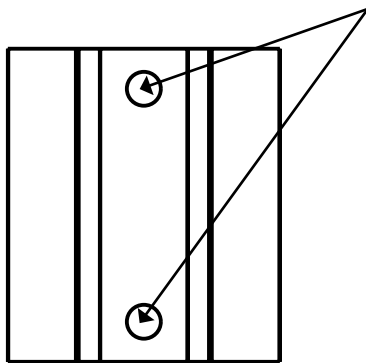
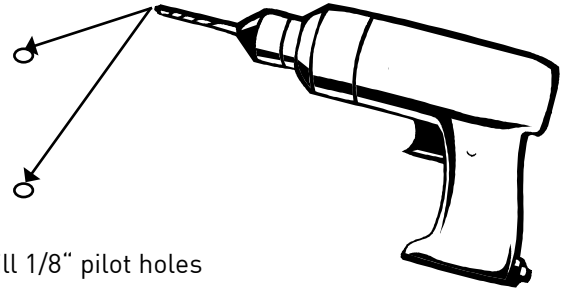
Taladre orificios piloto de 1/8"

**14**

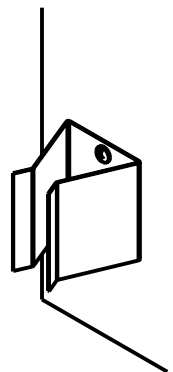Secure bracket with provided wood screws.

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**WARNING:**

TO PREVENT SERIOUS OR FATAL INJURY: FOR USE ONLY BY PERSONS WEIGHING LESS THAN 300 POUNDS. Consult a healthcare professional for advice and instructions specific to your needs before use. Contact CSI immediately at 1-800-882-0116 if any components are missing or damaged. Install in accordance with CSI's instructions. Do not modify this product in any way. Check grab bar periodically after installation to be sure all screws are secure. FOR QUESTIONS OR ASSISTANCE WITH INSTALLATION CONTACT CSI AT 1-800-882-0116.

**LIMITED LIFETIME WARRANTY:**

If this product fails due to materials or workmanship at any time during the life of the product, Creative Specialties International (CSI), a division of Moen Incorporated, will replace it free of charge, postage-paid. Simply contact CSI at 1-800-882-0116 for replacement information. This warranty does not cover products which have been abused, altered, damaged, misused, cut or worn. TO THE EXTENT PERMITTED BY LAW, CSI DISCLAIMS ALL OTHER IMPLIED OR EXPRESS WARRANTIES INCLUDING ALL WARRANTIES OF MERCHANTABILITY AND/OR FITNESS FOR A PARTICULAR PURPOSE.

**LIMITATION OF LIABILITY:** THIS WARRANTY IS YOUR SOLE REMEDY AND CSI SHALL NOT BE LIABLE FOR ANY DAMAGES, WHETHER DIRECT, INDIRECT, INCIDENTAL, SPECIAL, CONSEQUENTIAL, EXEMPLARY, OR OTHERWISE, INCLUDING LOST REVENUES AND LOST PROFITS, ARISING OUT OF ANY THEORY OR RECOVERY, INCLUDING STATUTORY, CONTRACT OR TORT. NOTWITHSTANDING THE TERM OF ANY LIMITED OR IMPLIED WARRANTY, OR IN THE EVENT THAT ANY LIMITED WARRANTY FAILS OF ITS ESSENTIAL PURPOSE, IN NO EVENT WILL CSI'S ENTIRE LIABILITY EXCEED THE PURCHASE PRICE OF THIS PRODUCT. SOME STATES AND PROVINCES DO NOT ALLOW THE EXCLUSION OR LIMITATION OF INCIDENTAL OR CONSEQUENTIAL DAMAGES SO THE ABOVE LIMITATIONS OR EXCLUSIONS MAY NOT APPLY TO YOU. THIS LIMITED WARRANTY GIVES YOU SPECIFIC LEGAL RIGHTS, AND YOU MAY ALSO HAVE RIGHTS WHICH VARY FROM STATE TO STATE AND PROVINCE TO PROVINCE.

For additional warranty information, visit [www.moen.com/homecare](http://www.moen.com/homecare)  
Grab bar meets or exceeds applicable requirements for the Americans with Disabilities Act.

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Home Care and Creative Specialties are registered trademarks of Moen Incorporated. Creative Specialties is a division of Moen Incorporated. © 2014