Kingsley Technical Specification

Grab Bar YG5400

Stock Numbers:

- q **YG5412CH 12" Grab Bar** Chrome Finish
- q **YG5412BN 12" Grab Bar** Brushed Nickel Finish
- q **YG5412ORB 12" Grab Bar** Oil Rubbed Bronze Finish
- q **YG5418CH 18" Grab Bar** Chrome Finish
- q **YG5418BN 18" Grab Bar** Brushed Nickel Finish
- q **YG5418ORB 18" Grab Bar** Oil Rubbed Bronze Finish
- q **YG5424CH 24" Grab Bar** Chrome Finish
- q **YG5424BN 24" Grab Bar** Brushed Nickel Finish
- q **YG5424ORB 24" Grab Bar** Oil Rubbed Bronze Finish
- q YG5430CH 30" Grab Bar Chrome Finish
- q **YG5430BN 30" Grab Bar** Brushed Nickel Finish
- q **YG5436CH 36" Grab Bar** Chrome Finish
- q **YG5436BN 36" Grab Bar** Brushed Nickel Finish
- q **YG5442CH 42" Grab Bar** Chrome Finish
- q **YG5442BN 42" Grab Bar** Brushed Nickel Finish

Materials:

Grab Bar posts and bars are aluminum. Skirt Rings are brass.

Compliance:

CSI grab bars meet or exceed the applicable requirements of the following standards:

ICC/ANSI A117.1

ASTM F446

CSA B45


CSA B651

Load Tested to withstand 300 lbs when securely installed per instructions.

12" long grab bars are intended for vertical installation only.



Dimensions:

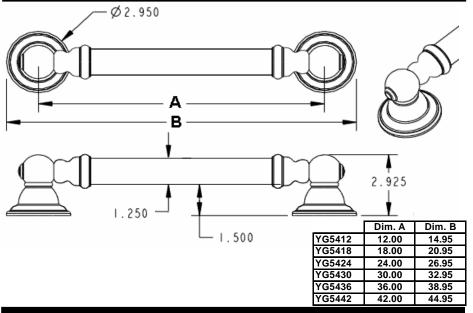

Installation Instructions:

Tools Required: 1/8" drill bit and drill, Phillips screwdriver, Tape and Level Hardware kit contents: 6 screws

- 1. Please note that all screws must mount into a wood stud.
- 2. Pull up and slide the skirt ring away from the hub end to expose the mounting screw holes on both hubs.
- 3. Locate the studs behind the wall by using a stud sensor or another stud locating method. Place the grab bar on the wall where desired so that all the mounting holes are positioned over a wood stud. Use a level if desired to verify placement.
- 4. With a pencil, mark all six mounting hole locations.
- 5. Remove the grab bar from the wall and drill 1/8" diameter pilot holes at the six marked screw locations. If desired, apply silicone caulk to the back of the grab bar hubs to guard against water penetration.
- 6. Mount the grab bar using all 6 of the #10 x 2 ½" long wood screws provided. Tighten screws until secure.
- 7. Slide the skirts back into place and press onto the hubs to secure.

Note: Grab bars are to be mounted to a secure surface. Creative Specialties International shall not accept responsibility for any and all damage caused by lack of wall strength or improper installation. It is the responsibility of the architect, builder and installer to ensure that the mounting area is secure and that proper mounting hardware is used for installation.

Cleaning Instructions:

To preserve the fine finish of this product, clean only with a soft damp cloth. Dry well. Do not use commercial or abrasive cleaners.

Stock Number:	Grab Bar
By CSI Bath Furnishings	

25300 AL MOEN DRIVE • NORTH OLMSTED, OH 44070 (800) 321-8809 • FAX: (800) 848-6636 • INTERNATIONAL: (440) 962-2000 • FAX: (440) 962-2726 CUSTOMER SERVICE (USA): 800-882-0116 ©Copyright CSI Bath Furnishing Division of Moen 2006 Literature #BA1125